


CARE & ATTENTION DURING CLINICAL USE



Review **corpuls cpr** settings.

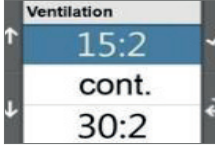

The **therapy settings** can be adjusted by pressing the **softkeys**.



Mark the stamp position on the sternum and review between rhythm checks.



If possible, **connect the charging cable** during use or when the battery LEDs indicate low remaining battery power.



Alternatively, press the two **red buttons** on the battery to remove it and **exchange for a charged battery**.

ADDITIONAL RESOURCES



corpuls cpr training page:
Further information and instructions



corpuls cpr Knowledge Hub:
Manuals, how-tos and guides



This card does not replace the thorough reading of the User Manual (P/N 09130.02)


UK Distributor:

Ortus Technology | Skyview Business Centre |
9 Churchfield Road | Sudbury | CO10 2YA

Phone +44 (0) 845 459 4705
E-mail hello@ortus.co.uk
Web theortusgroup.com



These guidelines are for in-hospital use with the Quadboard only. This pocket guide is for use in the United Kingdom. Availability depends on the regulatory and/or medical processes in individual markets. For availability please contact info@corpuls.com. Printing errors as well as construction and design modification subject to change. All mentioned product names are registered trademarks of the respective owners. Ver. No. 1.0 (02/25)

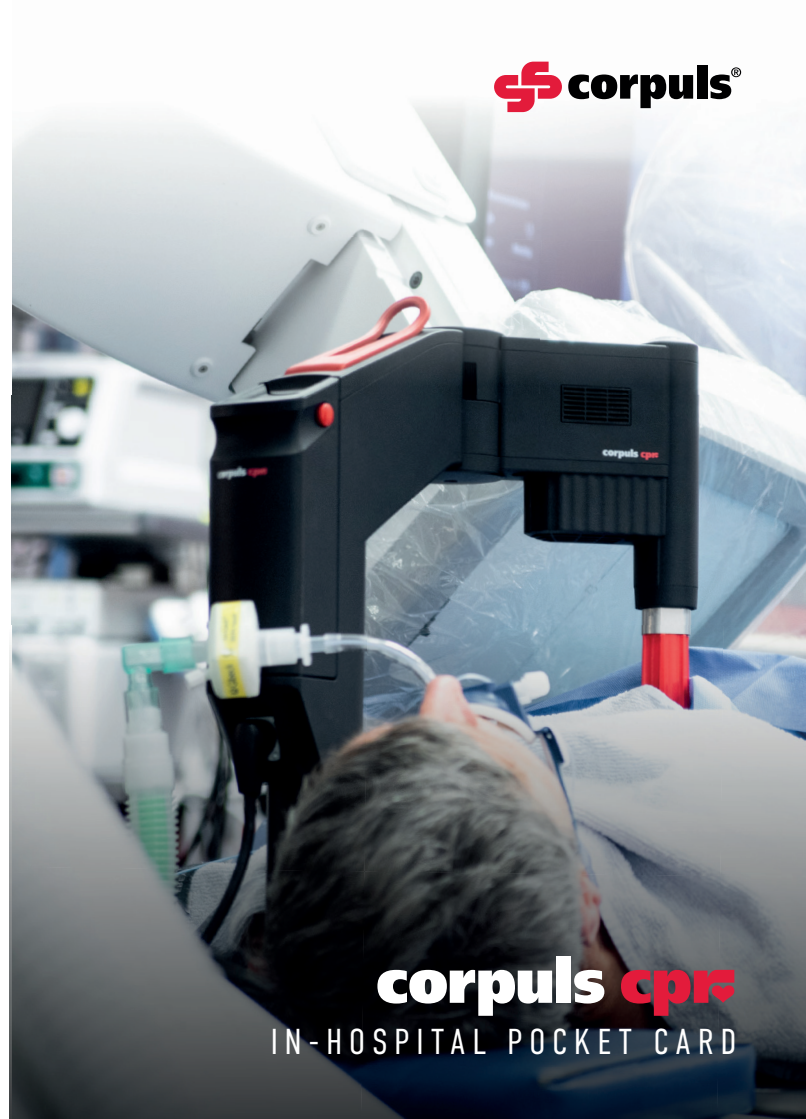

corpuls cpr
IN-HOSPITAL POCKET CARD

3-STEP APPROACH

1. POSITION THE QUADBOARD



1. Remove the Quadboard from the bag and **decide on the socket position** of the board.



2. Log roll or lift the patient by the shoulders to **slide the board under the patient**.



3. **Resume manual thorax compressions** with minimal interruption.

2. PREPARE CORPULS CPR



1. Remove from the bag & **power the device** on at the underside of the arm.




2. **Line up arm** vertically to board socket & **click into place**.
(If required, twist bayonet fitting)



3. Ensure the **correct stamp size** is in place.

3. ADJUST AND START

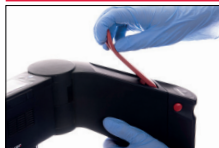

1. Release the locking lever and position the **corpuls cpr** stamp on the thorax. (Refer to **CORRECT STAMP POSITION**).



2. Lower the arm until the **stamp is touching skin** and the **green or amber LED** illuminates.



3. Use the palm of your hand to **lock the lever** and press the **start/stop button** to commence therapy.



IF RED LED ILLUMINATES, UNLOCK LEVER & RAISE ARM SLIGHTLY, UNTIL GREEN OR AMBER LED ILLUMINATES.




STAMP POSITION & ALARM NOTIFICATIONS

LED START/STOP Button	Symbol in the display	Description
		- Arm too low, position higher: unlock the lever and raise the arm slightly. - Release of the thorax not ensured. - Therapy not possible.
		- Arm too high, position lower. - If there is a visible contact between thorax and stamp, the user can still start therapy. - Configured compression depth not ensured throughout.
		- Arm adjusted optimally. - Full release and configured compression depth can be achieved.



ALARM MANAGEMENT

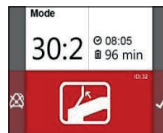


Press the softkey adjacent to the symbol to suspend audio-visual alarms for 2 minutes.

or

Correct an alarm condition and press the softkey adjacent to the symbol to confirm the audio-visual alarm alert.

CORRECT STAMP POSITION

- Position the stamp in the **centre of the lower part of the sternum**.
- Never** position the stamp **on or below the xiphoid process**.

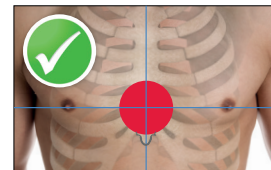

INTERRUPT THORAX COMPRESSIONS ONLY MINIMALLY!

